



# HAKUZO

## ORAL CARE CATALOG



*Hakuzo Medical Corporation*



ココ笑み®

HAKUZO NURSING CARE

# COCOEMI

Cocoemi is a brand created by Hakuzo  
for nursing care products.

We aim to provide products with high quality  
to fulfill the demands from not only the  
care receivers but also the caregivers.



Hakuzo Oral Care Series.

## What is oral care?

Oral care is to clean the oral cavity to maintain its functions, such as eating, speaking, laughing, breathing, etc..

Oral functions are important for having a better quality of life.



**Instructions (oral care should be at least performed 3 times per day)**

## Oral Care for Elderlies

Oral hygiene is important for elderlie's QOL.

The contaminants in oral cavity may raise the risk of aspirational pneumonia.

Cleaning the oral cavity can prevent aspirational pneumonia.

### ① Confirm the oral conditions

Check the oral cavity first.

### ② Moisturize the oral cavity For moisturizing the dry mouth and softening the plaque.

Moisturize the oral cavity to avoid damage the mucosa during the care. (P3 and P7)

### ③ Oral mucosa care For removing food residue and contaminants on oral mucosa and tongue.

Remove the contaminants like food residue and sputum from tongue and mucosa. (P4, P5, P6, P7, P8, P9)

### ④ Tooth brushing Mandatory for the elderlies with teeth.

Remove plaque on teeth. (P6)

### ⑤ Moisturizing the oral cavity Moisturize the oral cavity to finish the oral care.

Moisturize the oral cavity before finishing the care to prevent dry mouth. (P3 and P7)

**Denture cleaning (if needed).**

**Finished**

# Hakuzo Refreshing Oral Moisturizing Gel

Oral moisturizing gel for softening and removing plaque.  
Formulated with moisturizing sodium hyaruronate.



## Product Features

- Efficient in softening plaque.
- Thick texture.
- Easy to spread.
- Formulated with sodium hyaluronate.
- 2 flavors.
- Paraben free.

## Ingredients

Water, glycerin, PG, xylitol, saccharin Na, sodium hyaluronate, glycyrrhizic acid 2K, menthol, hydroxyethyl cellulose, xanthan gum, cetylpyridinium chloride, PEG-60 hydrogenation castor oil, citric acid, sodium citrate, polyphosphate Na, Na benzoate, fragrance.

Product Code	Specifications	Qty/box	Qty/carton
3901180	Honey & Lemon 80g	1 tube	60 boxes
3901181	Pine & Apple 80g	1 tube	60 boxes

## Instructions

Apply an adequate amount of the gel on an oral sponge or a toothbrush and clean the oral cavity. Spit out or wipe off the gel after cleaning. Can be used for oral massage as well.



Sweet and flavorful  
**HONEY & LEMON**



Refreshing and fruity  
**PINE & APPLE**



To moisturize oral cavity with refreshing flavors

Cosmetics

# Hakuzo Refreshing Oral Cleaning Wet Wipes

Wet wipes for oral hygiene and bad breath prevention

Plaque can be removed efficiently with uneven surface of the nonwoven fabric.


## Product Features

- Plaque can be removed easily with uneven surface of the wipes.
- Soft wipes made of 100% cotton is being used.
- High tensile strength.
- Efficient in softening and removing plaque.
- Formulated with moisturizing trehalose.
- Alcohol free.
- Refreshing mint flavor.
- Convenient portable size.

Green: Nonwoven fabric

Pink: Gauze


Cross-section view

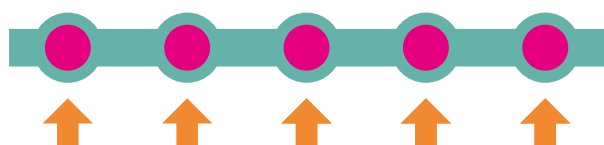


With the uneven surface, plaque can be removed efficiently.

## Instructions

Wrap the wipe around the finger, or fold the wipe and clean the oral cavity accordingly. Can be used for gum and mucosa massage and tongue cleaning.

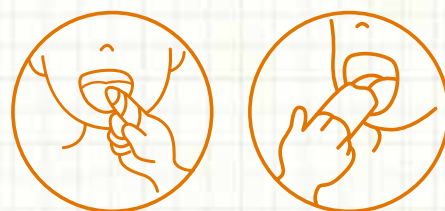

Cleaning oral cavity with refreshing mint flavor.

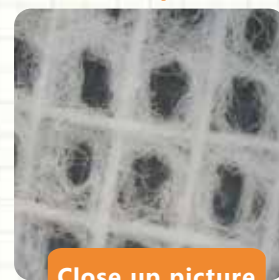

Close up picture

Hybrid fabric made of gauze and nonwoven fabric is being used.

## Ingredients

Water, PG, DPG, PEG-60, hydrogenated castor oil, glycerin, trehalose, xylitol, menthol, citric acid, sodium citrate, cetylpyridium chloride, methylparaben, propylparaben, glycyrrhizic acid 2K, acesulfame K, sucralose, fragrance.

Product Code	Specifications	Qty/pack	Qty/carton
1156993	150mm x 200mm Mint Flavor	50 pcs	40 packs


# Hakuzo Refreshing Oral Cleaning Dry Wipes

Dry wipes for oral hygiene and tongue cleaning.

Plaque can be removed efficiently with uneven surface of the nonwoven fabric.

## Product Features

- Plaque can be removed easily with uneven surface of the wipes.
- Soft fabric made of 100% cotton is being used.
- High tensile strength.
- Suitable size for tongue cleaning.

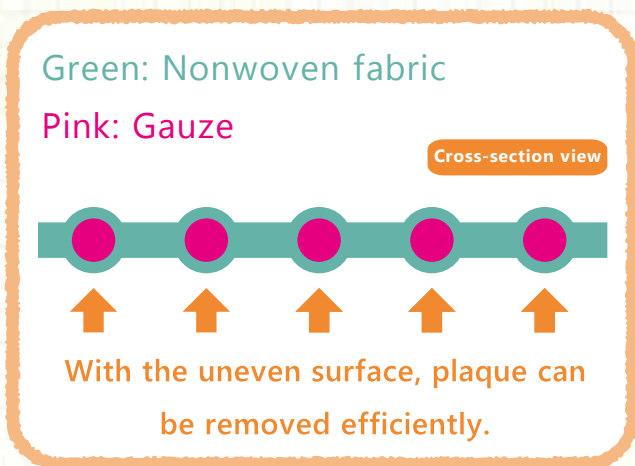


Product Code	Specifications	Qty/pack	Qty/carton
1156993	80mm x 200mm 2-fold	150 pcs	40 boxes

## Instructions

Wrap the wipe around the finger, or fold the wipe and clean the oral cavity accordingly. Can be used for gum and mucosa massage and tongue cleaning.



Can be taken out from the backside of the box.

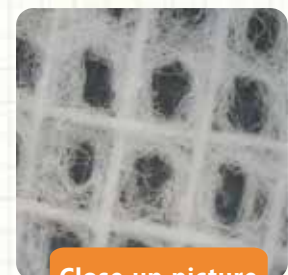

Close up picture

Hybrid fabric made of gauze and nonwoven fabric is being used.

# HAKUZO

## 360 Degree Toothbrush

360 degree bristle offers better handling for care givers.  
The plaque can be removed efficiently with capillary action.

### Product Features

- With the use of 24,000 bristles (20x of regular toothbrush), plaque can be removed efficiently with capillary action.
- ※ Capillary action is a phenomenon that liquid is sucked up between thin tubes by the action of surface tension of the liquid.
- Soft ultra thin bristle is being used.
- Round-tipped bristle is suitable for gum and mucosa cleaning.
- Can be used for tongue cleaning.



Product Code	Specifications	Qty/box	Qty/carton
3162360	BrushΦ 15mm White	10 pcs	50 boxes

Shaft Material	PP	Bristles Type	Soft
Bristles Material	Nylon	Heat Resistant Temperature	90°C

### Instructions

Do not rotate the brush. Brush the teeth like regular toothbrush.

Front teeth




Backside of the front teeth



Cuspid



The upper and lower front teeth

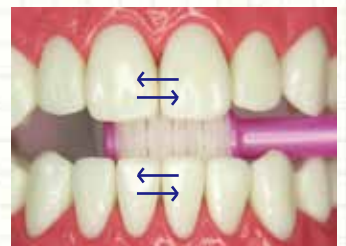

Back teeth



The upper and lower back teeth



Gum care



Tongue cleaning



# Hakuzo Refreshing Oral Cleaning Sponge

Suitable for ones can not do toothbrushing or gargling.

## Product Features

- Oral contaminants can be removed efficiently with specially-shaped sponge head.
- Can be used to apply moisturizing gel.
- Comes with both plastic shaft and paper shaft.



### Plastic Shaft

Waterproof shaft is suitable for using with water or cleanser, and for users who tends to bite the shaft or with much saliva.



### Paper Shaft

Firm paper shaft is easy to handle and suitable for removing dry plaque. Paper shaft is more environmentally friendly.




- Comes with notches on the sponge head to catch the food residues.
- Soft sponge head can conform to uneven surfaces to remove oral contaminants.
- Can be used for cleaning the mucosa and the tongue by rotating the sponge gently.

Product Code	Specifications	Qty/pack	Qty/carton
1240973	Regular paper shaft	300 pcs	6 boxes
1240976	Regular plastic shaft	300 pcs	6 boxes

## Instructions

- Clean the tongue and gum by rotating and moving the sponge forward and backward.

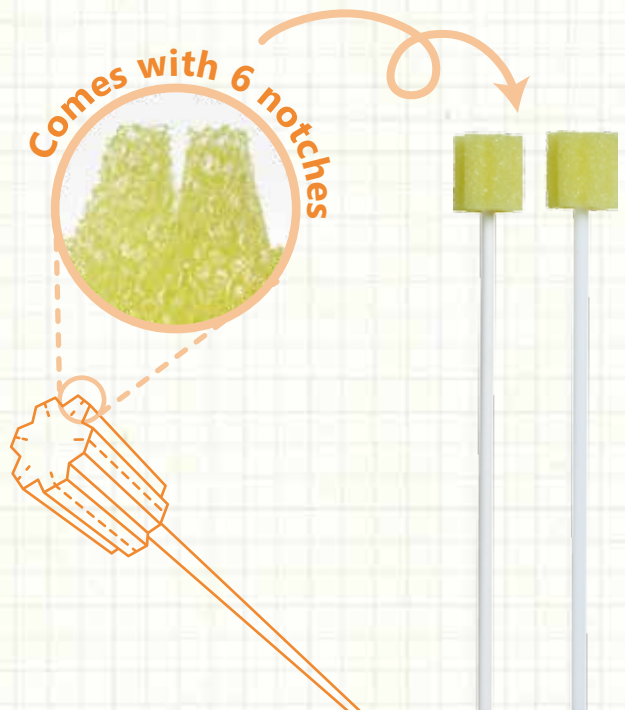



## Hakuzo Refreshing Oral Swab Half

Oral swab with cylindric cotton tip.

Product Code	Specifications	Qty/box	Qty/carton
1240973	paper shaft	200 pcs	20 boxes

### Product Features

- Thick and firm paper shaft is being used.
- Cylindric cotton tip is suitable for cleaning wide surface area like oral mucosa.



## Hakuzo Refreshing Oral Swab

Comes with both sponge tip and cotton tip. Suitable for cleaning the oral mucosa and the tongue.

### Product Features

- Zigzag sponge is suitable for cleaning the tongue.
- Cylindric cotton tip is suitable for cleaning wide surface area like oral mucosa.
- Thick and firm shaft is being used.
- Can be taken out without touching both tips. Hygienic and convenient.



Product Code	Specifications	Qty/box	Qty/carton
1240961	paper shaft	5 pcs x 20 packs	10 boxes

## Hakuzo Refreshing Oral Swab Spiral

Oral swab with spiral shaped cotton tip. Suitable for oral massage.

### Product Features

- Spiral shaped cotton tip is suitable for cleaning the oral cavity.
- Firm cotton tip hardly frays or comes off during the oral massage.
- Thick and strong paper shaft is being used.
- Can be used with moisturizing gel.
- With adequate absorption capacity, can be frozen for ice oral massage.



Spiral-shaped cotton tip

Product Code	Specifications	Qty/box	Qty/carton
3300039	spiral tip, paper shaft	1 pc x 50 packs	20 boxes

# Hakuzo Mouth Clean A

Oral swabs impregnated in solution. Ready to use.

## Product Features

- Impregnated in solution, can be used instantly.
- Refreshing flavors.
- Can be frozen and used for ice oral massage.



## 4 Merits

### No. 1

#### ○ Ready to Use

No preparation for water and cup needed. Suitable for situations lacking for access to water, such as home-visit nursing and bedside nursing.

### No. 2

#### ○ Low Risk of Aspiration

With adequate amount of solution, the risk of aspiration is minimized. Excess water must be squeezed out and wiped off with tissue when using the oral swabs impregnated in water.

### No. 3

#### ○ Refreshing flavors

Comes with refreshing fruity flavors that can let care receivers to enjoy the oral care. Stimulating the taste receptor can help improve the appetite.

### No. 4

#### ○ Suitable for Ice Oral Massage

Individually packed swabs are convenient and hygienic. Can be put in the freezer directly without the risk of being contaminated. Performing ice oral massage regularly can improve swallowing reflex.

Product Code	Specifications	Qty/box	Qty/carton
1290092	Lemon No. 12, 1 pc/pack	25 packs	40 boxes
1290093	Lemon No. 13, 1 pc/pack	25 packs	40 boxes
1290095	Blueberry No. 13, 1 pc/pack	25 packs	40 boxes

## Ingredients

Water, glycerin, sorbitol, sodium benzoate, peppermint oil, fragrance.

## Instructions

- ① Open the pouch and take out the swab.
- ② Clean oral cavity accordingly.



# Hakuzo Oral Care Products

Product Name	Product No.	Specifications	Packaging	Qty/carton	JAN Code
Hakuzo Refreshing Moisturizing Gel	3901180	Honey & Lemon 80g	1 Tube/box	60 boxes	4 958286 718095
Hakuzo Refreshing Moisturizing Gel	3901181	Pine & Apple 80g	1 Tube/box	60 boxes	4 958286 718101
Hakuzo Refreshing Oral Cleaning Wet Wipes	1156993	150mmx200mm Mint	50 pcs/pack	40 packs	4 958286 079967
Hakuzo Refreshing Oral Cleaning Dry Wipes	1156994	80mmx220mm 2 fold	150 pcs/box	40 packs	4 958286 718194
Hakuzo 360 Degree Toothbrush	3162360	BrushΦ15mm White	10 pcs/box	50 boxes	4 958286 718187
Hakuzo Refreshing Oral Cleaning Sponge	1240973	Regular/Paper Shaft	300 pcs/box	6 boxes	4 958286 717920
Hakuzo Refreshing Oral Cleaning Sponge	1240976	Regular/Plastic Shaft	300 pcs/box	6 boxes	4 958286 717937
Hakuzo Refreshing Oral Swab Half	1240012	Single Tip/Paper Shaft	200 pcs/box	20 boxes	4 958286 303154
Hakuzo Refreshing Oral Swab	1240961	Double Tip/Paper Shaft	5 pcs/pack 20 packs/box	10 boxes	4 958286 808000
Hakuzo Refreshing Oral Swab Spiral	3300039	Spiral Tip/Paper Shaft	50 pcs/box	20 boxes	4 958286 711843
Hakuzo Mouth Clean A	1290092	Lemon No. 12	25 pcs/box	40 boxes	4 958286 053080
Hakuzo Mouth Clean A	1290093	Lemon No. 13	25 pcs/box	40 boxes	4 958286 054001
Hakuzo Mouth Clean A	1290095	Blueberry No. 12	25 pcs/box	40 boxes	4 958286 058337



Design Your Voice

**Hakuzo**

Add: 2-4-9, Tokui-cho, Chuo-ku,  
Osaka 540-0025, Japan

TEL: 06-6942-0451

FAX: 06-6945-0539

[www.hakuzo.co.jp/eng/index.php](http://www.hakuzo.co.jp/eng/index.php)

Scan the QR code to access to our websites

Homepage



LINE



Instagram



Facebook

