

HAKUZO Clean&Refreshing Wet Towel

Individually packed disposable wet towel.



Cotton with soft texture is used.


- ◎For users with physical limitation.
- ◎Can be used for bottom cleaning when changing diapers.

HAKUZO Clean&Refreshing Wet Towel

2 sizes are available for different purposes.

L size

23.5cm×60cm

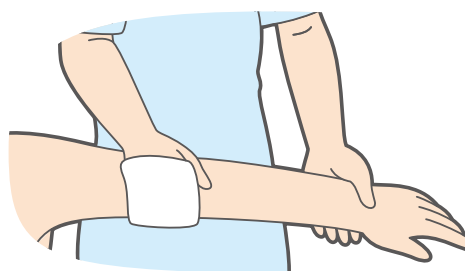

M size

30cm×30cm



Materials: cotton, rayon

Ingredients: water, benzalkonium chloride, iodopropynyl butylcarbamate

Product Features

The risk of infection can be decreased

Hygienic individual packaging. The risk of causing infection is lower than reused towels.

Thick cotton nonwoven fabric is used

The contaminants on skin can be wiped off easily without soiling hands.

Labor-saving

With the use of disposable wet towels, no hygiene management for reusable towels is needed.

※Do not warm up this product for long period of time. It may turn dry.


Purposes of Use

- ① To clean bodies of patients with physical imitations.
- ② To wipe face and hands.
- ③ To wipe off contaminants on skin.
- ④ To wipe corpses.
- ⑤ Suitable for travel, outdoors, emergencies, survival kits, etc.

Cautions on Use

- Do not flush down the toilet.
- Avoid contact with open wounds and mucus membranes.
- Stop using when skin is irritated.
- Use carefully when this product is heated up.

Product No.	Size	Specifications	Qty/bag	Qty/case	JAN code
3394081	L	23.5 cm×60 cm	50pcs/bag	10bags	4958286601335
3394082	M	30 cm×30 cm	100pcs/bag	8bags	4958286601328

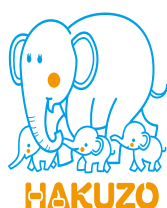

Design Your Voice

HAKUZO *Hakuzo*

Add: 2-4-9, Tokuicho, Chuo-ku,
Osaka 540-0025, Japan

TEL: 06-6942-0451

FAX: 06-6945-0539

www.hakuzo.co.jp/eng/index.php