

Container for disposing used needles and other sharps

Do Not Reuse



HAKUZO Sharps Container 3L

A safe and user-friendly sharps container to keep medical staff from being injured by sharp wastes.



To prevent injuries from sharp wastes...

It is mandatory to choose proper sharp wastes container based on the standards below.


- Container with resistance to penetration to prevent liquid leakage.
- Can be disposed in multiple ways to cope with different situations.
- Can be sealed completely when disposing to make sure the wastes do not fall out in any condition.

Can be used with customized holder.




Customized Container

Product Features

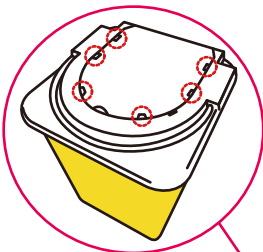

Safe to Use

The container can be temporarily closed securely and opened easily with 6 temporary closures.

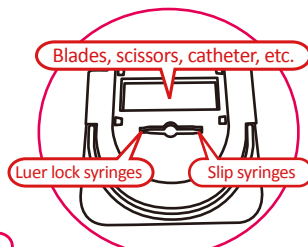

Comes with Needle Remover

Can be used to removed needles from both luer lock tip and slip tip syringes.

Maximum Filling Level Indicator

The risk of overfilling is largely decreased.

Blank Columns

The name of hospital, date of use, etc. can be written on the container.



Easy to Carry

Comes with handle and can be carried easily.

Resistance to Penetration

Made of heavy-duty polypropylene that has superior resistance to penetration and can be burned.




Instructions for Use

※Used sharps with no blood or fluid attached also need to be handled as biohazardous wastes and disposed properly.

How to Assemble




Attach lid to base container firmly and press down. Ensure you hear a click sound. Write down the name and date of assemble.

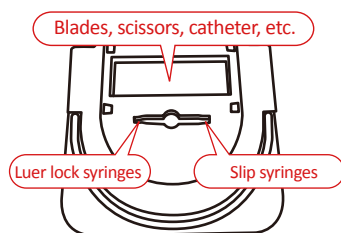


How to Dispose

① Butterfly Needles

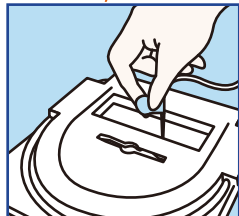


Hold the wings and drop it into the rectangular opening.

② Blades, Scissors, Catheters

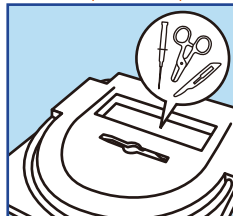


When disposing a needle with syringe, drop it into the rectangular opening.

③ Luer Lock Tip

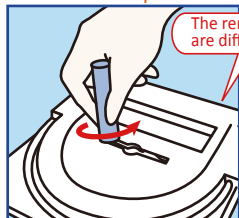

Insert the needle in the hole in the center, position the needle in the luer lock removing side and rotate the syringe to remove the needle.

④ Slip Tip

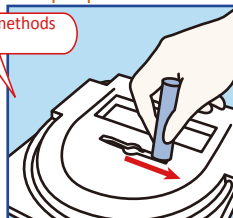


Insert the needle in the hole in the center, slide the syringe to the slip tip removing side, and the needle will fall off.

The removing methods are different.


How to Close

Temporary Closure

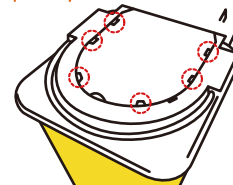

Press all 6 temporary closures and ensure to hear the click sounds to close the container temporarily.

Final Closure

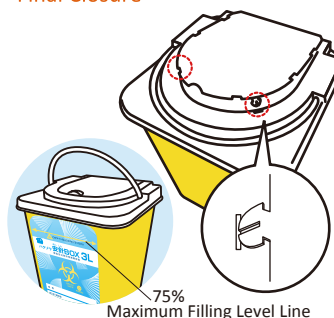

When the volume of wastes reaches the maximum filling level indicator, bend the small nails on all 6 closures, insert them into the small holes and ensure all closures are engaged with click sounds. Dispose the container.

Prohibition

- Do not shake the container.
- Do not overfilling. The sharp wastes may fall out from the container.

How to Carry

- Hold the handle to carry.
(Hands may be injured by penetrating needles when holding container directly.)



Product No.	Product Name	Capacity	Qty/Bag	Qty/Carton	JAN Code
3118051	Hakuzo Sharps Container	3L	5 pcs	6 bags	 4 958286 660011



Design Your Voice

Hakuzo

Add: 2-4-9, Tokuicho, Chuo-ku,
Osaka 540-0025, Japan

TEL: 06-6942-0451

FAX: 06-6945-0539

www.hakuzo.co.jp/eng/index.php