



Design Your Voice

**Hakuzo**

Latex Free

Non Sterile

HAKUZO Powder Free

# NITRILE GLOVES ES

200 pcs

Well fitting nitrile gloves  
with high chemical resistance and elasticity

Made from latex-free material.  
Can be used by latex allergy users.



Hakuzo Medical Corporation

# HAKUZO Powder Free NITRILE GLOVES ES

## Product Features

- High stretchability provides superior fit and comfort.
- Non-stock and anti-slip coatings give better handling.
- High chemical resistance.
- Made of latex-free material, can be used by latex allergy users.
- Powder free gloves minimise the risk of contamination from powder splash.



**Material** ● Nitrile Rubber


### Specifications

Size	Full Length	Palm Width
XS	240mm ± 10mm	75mm ± 5mm
S	240mm ± 10mm	85mm ± 5mm
M	240mm ± 10mm	95mm ± 5mm
L	240mm ± 10mm	110mm ± 5mm


### Quality Standards

Pinhole	AQL1.5
---------	--------

## How to Wear

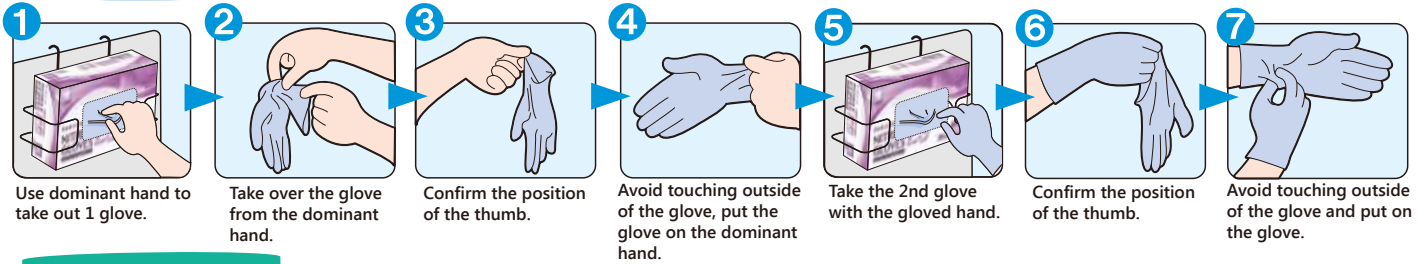

## How to Take off

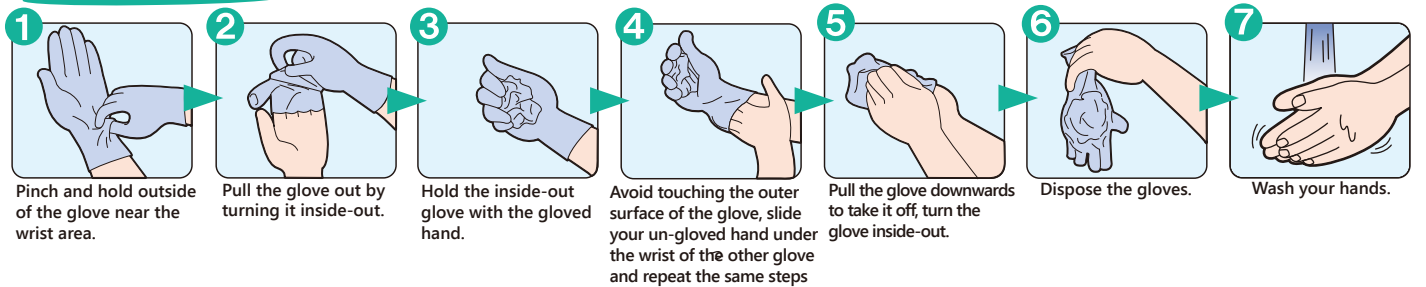

## Chemotherapy Drugs Permeation Testing



The following tests are based on the standard of ASTM D6978-05.



Test Method	Chemotherapy Drugs	Concentration	Breakthrough Detection Time
ASTM D 6978-05	Methotrexate	25.0mg/ml (25,000 ppm)	No breakthrough up to 240 minutes
	CDDP	1.0mg/ml (1,000ppm)	
	CPA	20.0mg/ml (20,000ppm)	
	Dacarbazine	10.0mg/ml (10,000ppm)	
	Doxorubicin hydrochloride	2.0mg/ml (2,000ppm)	
	Etoposide	20.0mg/ml (20,000ppm)	
	5-fluorouracil	50.0mg/ml (50,000ppm)	
	Paclitaxel	6.0mg/ml (6,000ppm)	
	Vincristine sulfate	1.0mg/ml (1,000ppm)	
	Mitomycin c	0.5mg/ml (500ppm)	

## Chemical Resistance Test

The following test is based on the standard of EN374-3.

Test Method	Chemical	Concentration	Breakthrough Detection Time
EN374-3:2003	Sodium hydroxide	40%	No breakthrough up to 480 minutes

Product No.	Size	Pcs / Box	Box / Case	JAN Code
3024071	XS	200pcs	12box	 4 958286 706337
3024072	S	200pcs	12box	 4 958286 706344

Product No.	Size	Pcs / Box	Box / Case	JAN Code
3024073	M	200pcs	12box	 4 958286 706351
3024074	L	200pcs	12box	 4 958286 706368



Design Your Voice

**Hakuzo**

Add: 2-4-9, Tokuicho, Chuo-ku, Osaka 540-0025, Japan

TEL: 06-6942-0451

FAX: 06-6945-0539

www.hakuzo.co.jp/eng/index.php