



With 0.1w/v% of Sodium Hypochlorite

Hakuzo Sodium Hypochlorite 1000ppm Wipes

Flushable type

Filmed Sodium Hypochlorite solution and 5 pieces of nonwoven fabrics are packed together.



With 0.5w/v% of Sodium Hypochlorite


Hakuzo Sodium Hypochlorite 5000ppm Wipes

Flushable type

Filmed Sodium Hypochlorite solution and 5 pieces of nonwoven fabrics are packed together.



Product Information

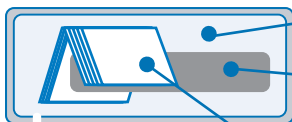

Outer packaging


Sodium Hypochlorite solution

Content: 0.1w/v% of Sodium Hypochlorite
/0.5w/v% of Sodium Hypochlorite
Solution amount: 45ml

Cleaning cloths

Size : 200mm x 300mm
Qty : 5 pcs


How to use

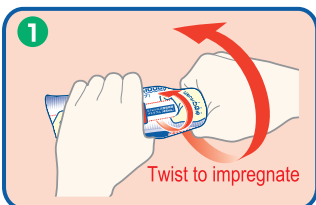

1

Twist to impregnate

Break the sealed Sodium Hypochlorite solution, impregnate the nonwoven fabrics evenly.

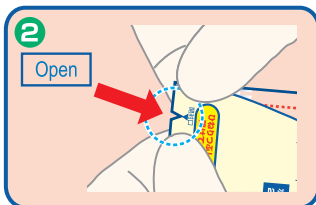

2

Open

Open the packet and take out the impregnated nonwoven fabrics.

※ Can just flush in the toilet.



※ Holder for this product can be purchased separately.

When to use

Please do not use for other purposes other than cleaning the environment

- To clean and sanitize tables, door knobs, bed surroundings and etc.
- To clean and sanitize places that are contaminated.

Attention

※ Please read the manual carefully before using.
People who are not feeling well, who have heart or breathing problems are not allowed to use this product.

- Please keep out of reach of children.
- Please do not use for other purposes other than the mentioned ones.
- Please avoid the solution from entering the eye.
- It might cause the clothes colors to fade.
- Please do not use it on other containers.
- Please break the seal before opening the packaging.
- Please put on rubber gloves when using.

Product Features


1 Simple and convenient

- ◎ Just break the sealed Sodium Hypochlorite solution, impregnate the nonwoven fabrics and open the packaging to be used.



2 High efficiency

- ◎ The concentration has been fixed and no other preparations are needed.

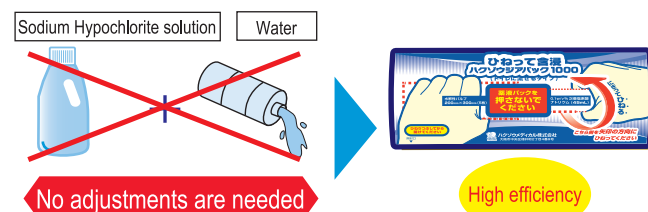

3 Can be kept for a long period of time

- ◎ High barrier packaging films are used, hence the concentration can be maintained for 2 years.
- (※ Please use as soon as possible once the nonwoven fabrics have been impregnated.)



4 Safe

- ◎ Sodium Hypochlorite solution are packed by a seal and also an outer packaging, minimizing the leaking problems.
- (※ Please put on rubber gloves when using this product.)

- Please avoid direct sunlight and store it in a clean place.
- This product might erosion to some metals.
- Please stop using cause immediately if discomfort occurs.
- Please flush only 1 or 2 pieces at a time.
- Please do not press on the sealed Sodium Hypochlorite solution.
- Please stop using if there is a leakage.



Please do not press on the sealed Sodium Hypochlorite solution!

Product Code	Specifications	Packaging	Qty/case	JAN Code
3162102	0.1w/v% of Sodium Hypochlorite hydrolyzable pulp 5 pcs (200mm x 300mm)	5pcs x 10 sets/box	20 boxes	 4 958286 705705
3162105	0.5w/v% of Sodium Hypochlorite hydrolyzable pulp 5 pcs (200mm x 300mm)	5pcs x 10 sets/box	20 boxes	 4 958286 706924



Hakuzo Medical Corporation

4-9, 2-chome, Tokuicho, Chuo-ku, Osaka
540-0025, Japan
Tel: 81-6-6809-4319 Fax: 81-6-6945-0539
E-mail: hmc-trade@hakuzo.co.jp
http://www.hakuzo.co.jp/eng/