

MADE IN JAPAN

MEDICAL DEVICE

SINGLE USE


Dr. HUBY[®]

TROCAR CLEANER

For Medical Use

EO STERILE

Dr. Huby Trocar Cleaner is a cotton tipped swab designed for cleaning trocar and camera lens.

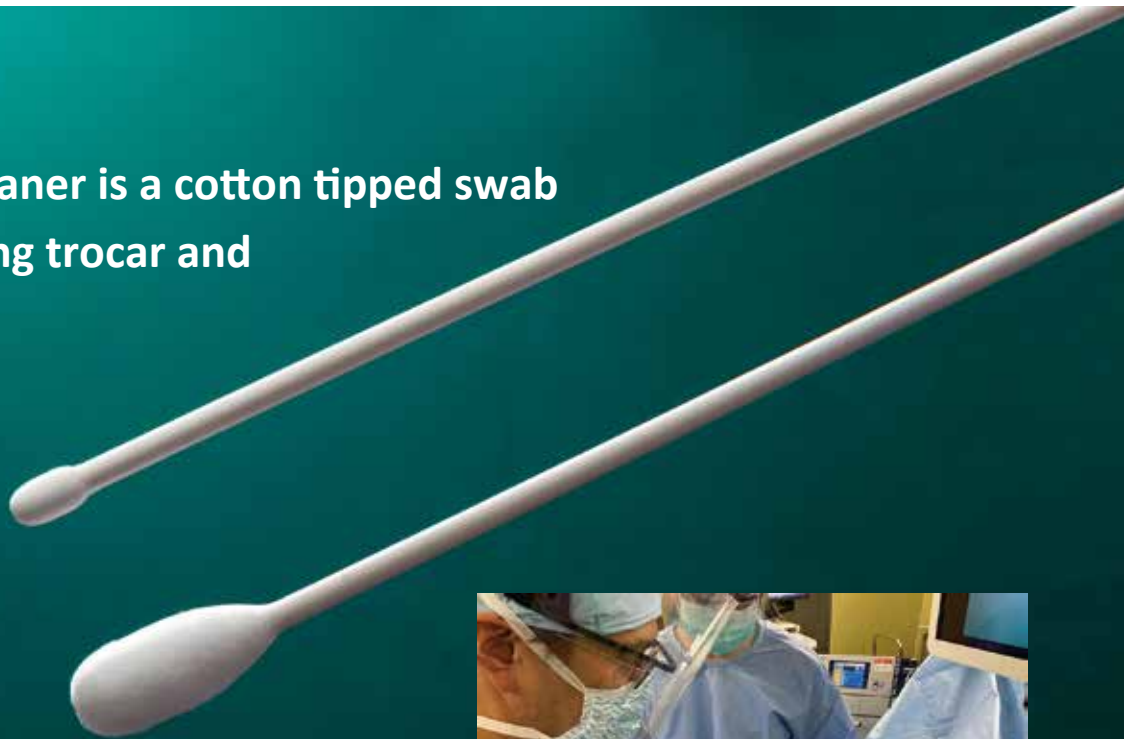

Convenient double tips



The photo is provided by Next Generation Endoscopic Intervention of Department of Surgery, Osaka University


This product is jointly developed with
Graduate School of Medicine/Faculty of Medicine, Osaka University and
Sanyo Co., Ltd.

Hakuzo Medical Corporation

■ Concept

Soiled trocar cannula may contaminate the camera and worsen the surgical field during endoscopic surgery. Currently gauze is commonly used for cleaning the trocars but it is not easy to use. Therefore, the demands of cleaning tools for trocar cannula are increasing.

In collaboration with Osaka University, we have developed a cotton swab that allows smooth cleaning of trocars.

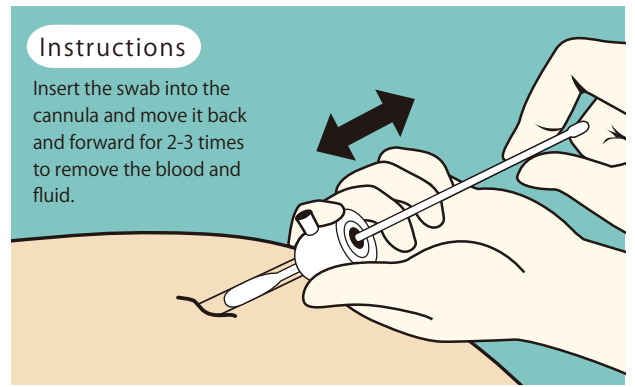

Instructions

Insert the swab into the cannula and move it back and forward for 2-3 times to remove the blood and fluid.

■ Product Features

• Medical Device

This product is the first trocar cleaner that is classified as medical device in Japan. We have run various bench tests and biological safety tests on this product to ensure its safety and efficiency.

• 2 Sizes of Cotton Tips

1 swab comes with 2 sizes of tips to cope with different types of trocars, One tip can be used to clean the trocar, the other tip can be used to clean scope lens and there is no waste.

• Superior Absorbency

Cotton with superior absorbency is being used. The swab can absorb any fluid and remove debris from the trocar cannula.

• Lint Free Cotton tips

The cotton tips are coated with glue and the risk of shedding fibers is minimized.

• Firm Shaft


The shaft is made from paper and it is easy to hold and use.

• Individually Packed Swabs

Individually packed swabs are hygienic and easy to use.


• Other Features

- EO Sterile ※The shelf life is unopened up to 3 years.
- Conforms to Safety Test (not GLP)

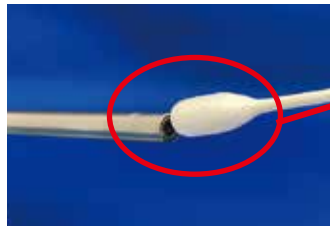

The unused cotton tip can be used to clean camera lens.

The photo is provided by Next Generation Endoscopic Intervention of Department of Surgery, Osaka University

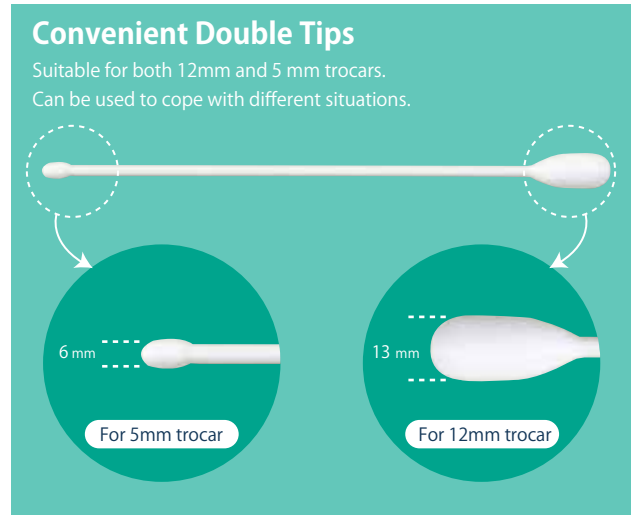


Convenient Double Tips

Suitable for both 12mm and 5 mm trocars. Can be used to cope with different situations.

■ Specifications

【Materials】 Cotton Tips: absorbent cotton Shaft: paper Glue: polyvinyl alcohol, glycerin 【Weight】 5g per stick

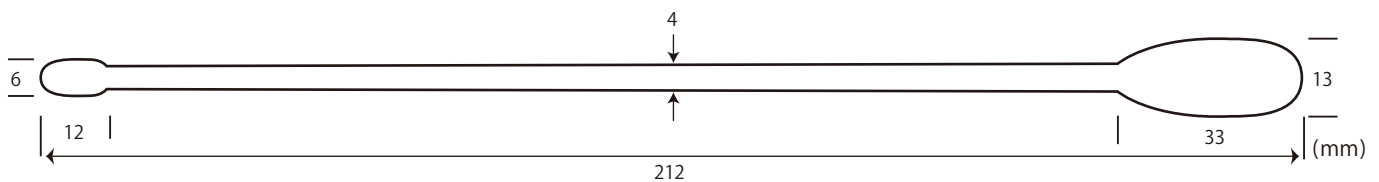

Product Code	Product Name	Specifications	Qty/box	Qty/carton	JAN Code
1240618	Dr.HUBY Trocar Cleaner	1stick/pack (Φ 6mm×212mm) (Φ13mm×212mm)	20 packs	16 boxes	4 958286 078892

Package Insert QR Code



Design Your Voice

Hakuzo

Add: 2-4-9, Tokui-Cho, Chuo-ku, Osaka 540-0025

TEL: +816-6809-4319

FAX: +816-6942-0496

www.hakuzo.co.jp