

- Thin dressing pad for absorbing blood and fluid.
- Comes with moisture permeable waterproof film.

Medical Device

Hakuzo Cure Dressing S

EO Sterile




Purposes

To absorb blood and fluid.
To protect wounds.

Product Feature① Thin dressing pad.

7 layer thin dressing pad.

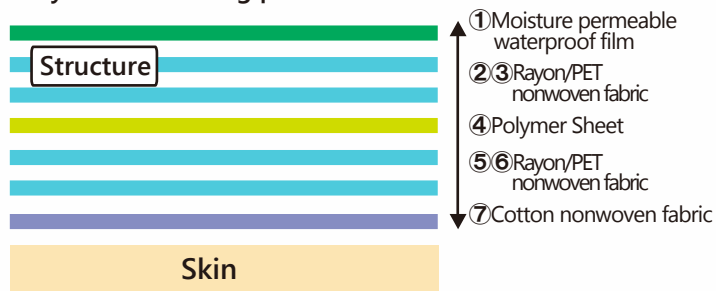


Product Feature② High absorbency.

Polymer with superior absorbency is being used.
Suitable for absorbing blood and fluid.

15cm×15cm (About 6g/pc) About 200g	15cm×30cm (About 12g/pc) About 320g
26cm×30cm (About 20g/pc) About 650g	


※the absorbent volume is measured based on "The Standards of Medical Absorbent Cotton"

Product Feature③ Beige film.

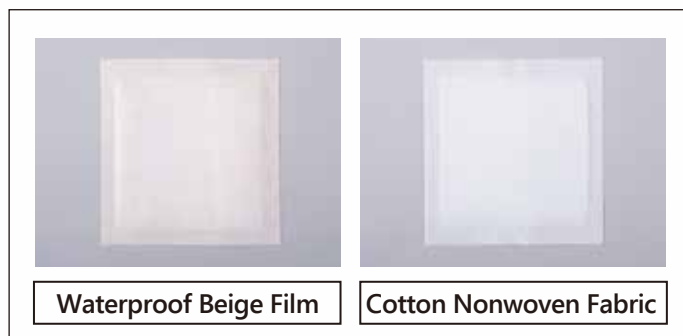

Product Feature④ Moisture permeable dressing.

Moisture permeable waterproof film provides suitable water retention and minimizes the risk of skin rashes.

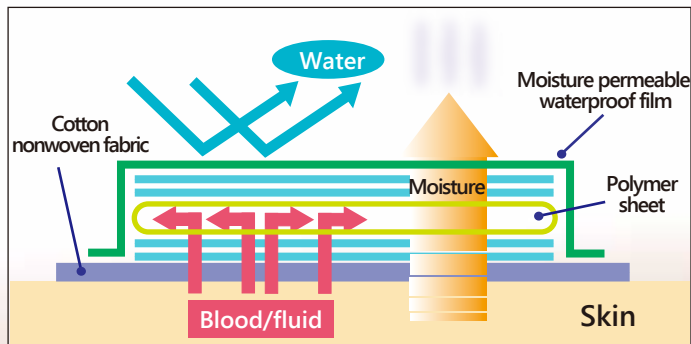

Hakuzo Medical Corporation

Hakuzo Cure Dressing S

Do not reuse

[Prohibition]

- Do not reuse

[Materials]

Cotton nonwoven fabric: absorbent cotton

Absorbent sheets: rayon, PET, pulp, polymer

Waterproof film: polyethylene, polypropylene

Adhesive: hot-melt adhesive

[Purposes]

To absorb blood and fluid and protect wounds.

[Instructions]

1. choose the dressing that is bigger than the wound. Take out the dressing from the pouch and applied the absorbent side(white) on the wound.
2. Fix the dressing firmly with tapes.

[Cautions]

1. Basic cautions

- (1) Do not cut this product.
- (2) External use only.
- (3) Stop using this product and take proper action if infection or skin lesion occurs.
- (4) Pay attention to the condition of the wound when using this product. Change the dressing accordingly.
- (5) Avoid applying pressure on the wound.

(6) Do not apply the waterproof film on the wound.

(7) Use immediately after opening.

2. Disorder/Adverse events

The symptoms below may occur during the use of this product.




- Infection.
- Skin lesion (exfoliation, maceration, edema, blisters, skin redness, erosion, itching, allergies, contact dermatitis.)
- Adhesion
- Increase in necrotic tissue
- Sore




[Storage]

1. Avoid direct sunlight and wet and hot environment. Store in a clean and cool place.
2. Expiry date is printed on the package. The shelf life is set based on the in-house test results.

[Manufacturer]

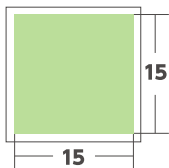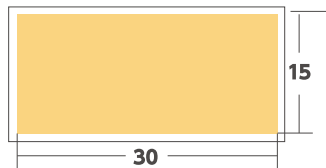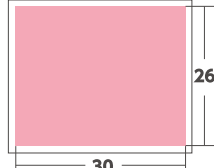Hakuzo Medical Corporation
4-9, 2 Chome, Tokui-cho, Chuo-ku
Osaka 540-0025 Japan
<http://www.hakuzo.co.jp/>
TEL 06-6809-4319

Size	15cm x 15cm	15cm x 30cm	26cm x 30cm
Image			
Weight (without pouch)	About 6g/pc	About 12g/pc	About 20g/pc

Product Code	Size	Qty/Box	Box Size	Qty/Carton	Carton Size	JAN Code
3036100	15cm x 15cm	1 pc x 30 bags	W260×D140×H140mm	4 boxes	W535×D295×H155mm	
			About 400g		About 2168g	
3036101	15cm x 30cm	1 pc x 30 bags	W260×D215×H140mm	4 boxes	W535×D445×H155mm	
			About 676g		About 3422g	
3036102	26cm x 30cm	1 pc x 15 bags	W220×D215×H140mm	4 boxes	W455×D445×H155mm	
			About 494g		About 2700g	



Design Your Voice

Hakuzo

Add: 2-4-9, Tokui-Cho, Chuo-ku,
Osaka 540-0025

TEL: +816-6809-4319

FAX: +816-6942-0496

www.hakuzo.co.jp