



High-absorbency wound protecting pad



HAKUZO

STERILISED WRAP CURE DRESSING

EOG sterilized high-absorbency wound protecting pad, comprising non-adhesive films with high absorbency cotton.

50 Packets per box



50mm x 50mm

100mm x 100mm

Product features

How to use



1 Tear open the perforated part and pull out 1 packet of HakuZo Sterilized Wrap Cure Dressing.



2 Open the sterilized packaging and put the pad on the wound.

When to use

◎When there is a minor bleeding.


※Not suitable to be used for serious bleeding.

1 Non-adhesive films are used

On the surface of the pad, non-adhesive polyethylene films are used.

2 Moisture environment

High breathability polyethylene films are used, blood and other body fluids are properly absorbed and the moisture level is properly maintained.

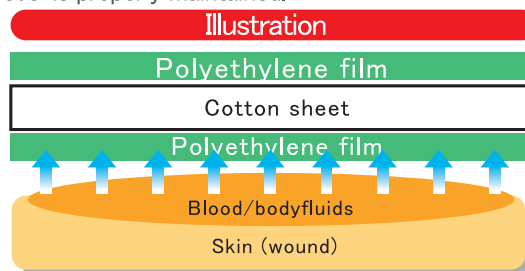

3 Sterilised

Pads are EOG sterilized piece by piece, guaranteeing the hygiene. There are 2 sizes to choose from.




HAKUZO

STERILISED WRAP CURE DRESSING

【Don'ts】
 · Please do not reuse

【Structure】

Pad: Cotton sheet is placed in between 2 Polyethylene films

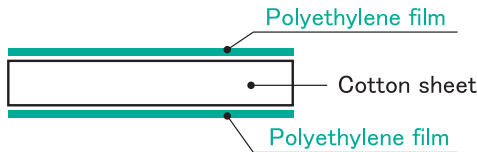

【Purpose of usage】

To protect wounds

【How to use】

Clean the wound properly, put the pad on the wound and stabilize it with an adhesive bandage.

【Attention】

- ① Please change a new pad when the pad gets contaminated or wet.
- ② Please use only once.
- ③ Please stop using if rashes, rubefaction or itching occurs.
- ④ This is a sterilized product, please use immediately once opened.
- ⑤ Please stop using if expired.

【Storage】

《 Storage instructions 》
 Avoid direct sunlight, keep out of children.

【Packaging】

50mm X 50mm 50 packets per case
 100mm X 100mm 50 packets per case

Product Code	Picture	Specifications	Packaging	Qty/case	JAN code
3155131		50mm × 50mm	50 pckets/box	12 boxes	4 958286 705576
3155132		100mm × 100mm	50 pckets/box	12 boxes	4 958286 705583

HAZKUZO High-absorbency wound protecting pad
Wrap Care Dressings

Non-sterilised



1 roll
 75mm × 5m



1 roll
 150mm × 5m



Roll type, can cut the size according to the size of the wound.



A printed ruler is available on the box

Non-sterilized types are also available

Roll type, can cut with desired length



Hakuzo Medical Corporation
 4-9, 2-chome, Tokuicho, Chuo-ku, Osaka
 540-0025, Japan
 Tel: 81-6-6809-4319 Fax: 81-6-6945-0539
 E-mail: hmc-trade@hakuzo.co.jp
<http://www.hakuzo.co.jp/eng/>