

Single Use



Medical Device

Calcium Alginate Hemostatic Pads

Hakuzo ALG Pads

- For post-dialysis hemostasis.
- For patients having problems stopping bleeding.

EOG
Sterile




【Size】

● 2cm × 2cm

【Materials】

- Waterproof film: Polyethylene
- Nonwoven fabric: Calcium alginate
- Mesh film: Polyethylene

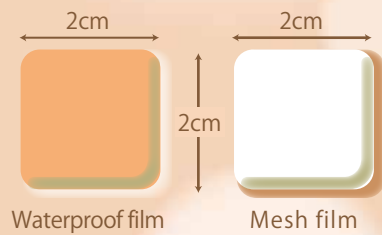

【Cross Section】



What is Calcium Alginate?


Calcium alginate fibers that is made from alginic acid extracted from kelp and wakame seaweed, can absorb blood and fluid to form gel on the wound.

Product Features


1. Calcium alginate nonwoven fabric is suitable for patients having problems stopping blood.
2. Comes with waterproof film on the top to prevent leakage of blood.
3. Comes with mesh film to prevent recurrent bleeding when removing the pad.
4. Hygienic individual package.




Instructions

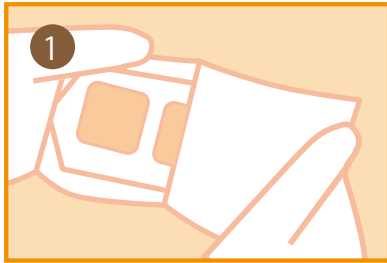

① Take out 1 pad from the pouch.

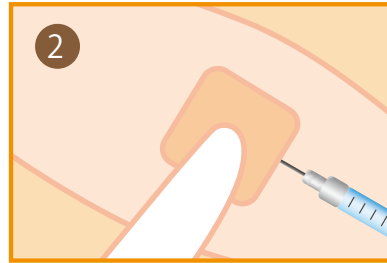

② Place the mesh film side on the puncture wound. Press the pad down and remove the needle.

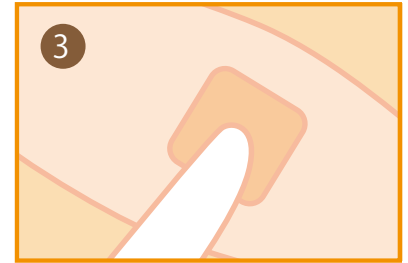

③ Press down the pad until the bleeding is stopped. Check the puncture wound during the hemostasis accordingly.

Hakuzo ALG Pads

Do not reuse

【Prohibition】

• Do not reuse.

【Description】

This product is an EO sterile 3-layer pad made of polyethylene mesh film, calcium alginate nonwoven fabric and polyethylene waterproof film.

【Purposes】

For stopping bleeding at the catheter insertion site in the thigh or forearm (radial artery) after angiography and other diagnostic procedures.

【Instructions】

Take out 1 pad without contaminating it and place the mesh film side on the puncture wound. Press down the pad until the bleeding is stopped or fix it with tape or bandage.

【Cautions】

- 1) Use this product immediately after opening the pouch.
- 2) Stop using this product if any irritation, itchiness, or redness develops.

【Storage and Expiry Date】

1. Store this product in a clean place and avoid water, direct sunlight, high heat and wet environment.

2. The expiry date is printed on the package.

[The shelf life is set based on in-house researches]

【Manufacturer and Distributor】

• Manufactured and distributed by
Hakuzo Medical Corporation
4-9, 2 Chome, Tokui-cho, Osaka-shi,
Osaka, 540-0025 JAPAN
<https://www.hakuzo.co.jp/>
TEL 06-6809-4319

Product Code	Specs	Qty/box	Box Size (mm)	Qty/carton	Carton Size (mm)	JAN Code	GS1 Code
			Box weight (g)		Carton Weight (kg)		
1290200	2 cm × 2 cm 2 pcs/pouch EO Sterile	100 pouches	W265×D89×H65	20 boxes	W455×D272×H317	4 958286 080680	(01) 14958286080687
			150		3.5		



Design Your Voice

Hakuzo

Add: 2-4-9, Tokui-cho, Chuo-ku,
Osaka 540-0025, Japan

TEL: 06-6942-0451

FAX: 06-6945-0539

www.hakuzo.co.jp/eng/index.php