


ELEBAN BLOCK



Waterproof Island Dressing

MEDICAL DEVICE

sterilized with EO

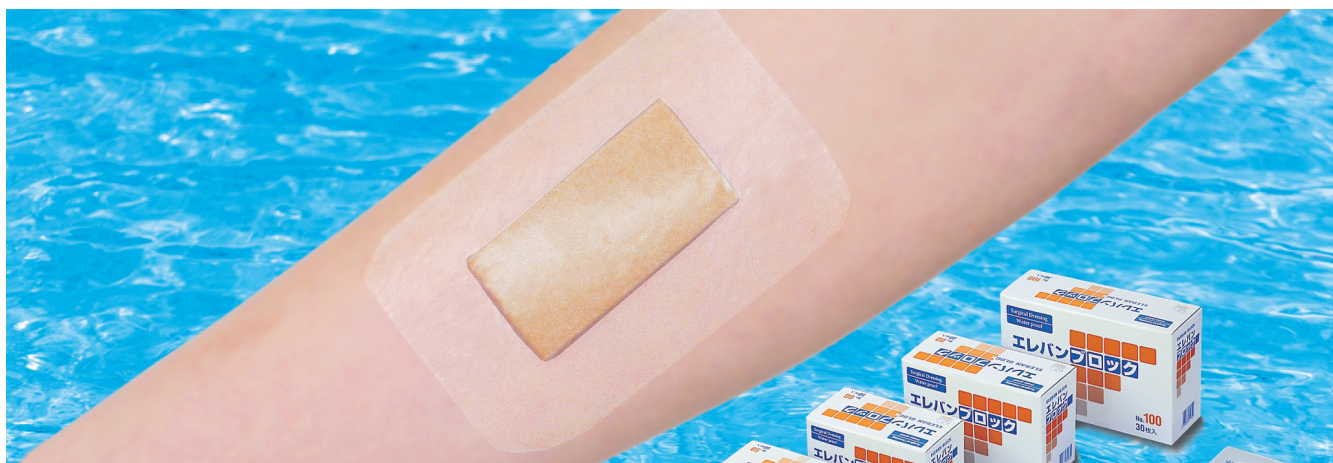

IEEBAN BLOCK is a waterproof island dressing, using transparent film made from polyurethane.

Polyurethane film


- Permeable to oxygen and moisture vapor, but impermeable to bacteria and fluid
- Easy application and excellent at fixing

Nonwoven pad

- Has a rayon pad covered with meshed film for preventing adherence to the wound

Rayon-made nonwoven pad

Meshed film for preventing adherence

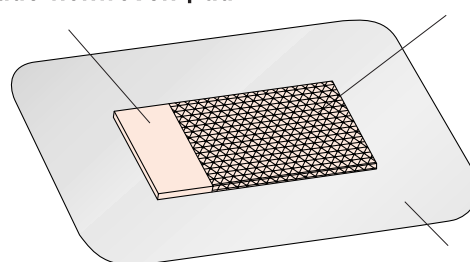

Polyurethane film

●How to use

<p>1</p> <p>Peel off the larger release paper on the reverse side.</p>	<p>2</p> <p>Peel off the other release paper while applying Eleban Block to the wound or puncture site. Make sure that film is applied to the skin securely.</p>	<p>3</p> <p>Peel off one auxiliary film.</p>	<p>4</p> <p>Peel off the other auxiliary film.</p>
---	---	---	---

Product	Size (Pad)	Pcs/box	Boxes/carton
No.50	5cm × 7cm(2cm × 4cm)	50	20
No.60	6cm × 9cm(3cm × 6cm)	50	20
No.80	8cm × 12cm(4cm × 8cm)	30	20
No.100	10cm × 15cm(5cm × 11cm)	30	20

MADE IN JAPAN



Hakuzo Medical Corporation

4-9, 2 chome, Tokuicho, Chuo-ku, Osaka

540-0025, Japan

Tel; 81-6-6809-4319 Fax; 81-6-6945-0539

E-mail; trade@hakuzo.co.jp