



ELEBAN WIDE DX

Non-Woven Island Dressing

EO Sterile



Product Features

Absorbent Pad

Cotton with superior absorbency is being used. The non-adherent surface keeps the pad from sticking to the wound.


Non-Irritating Adhesive Tape

Almost allergy free acrylic adhesive is being used. Special process on the adhesive provides superior breathability.

Water Repellent Non-Woven Fabric

The non-woven fabric is water repellent and can keep the wound from being contaminated.

Comes with non-adherent film to keep the pad from sticking to the wound.

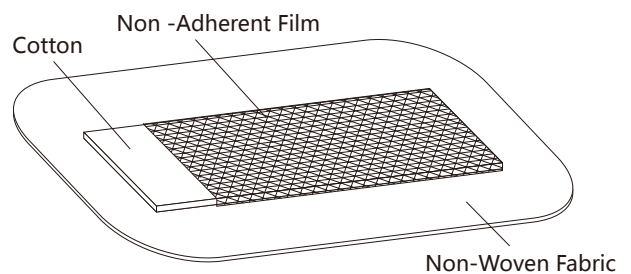

ELEBAN WIDE DX

Non-Woven Island Dressing

Eleban Wide DX

[Prohibition]
Do not reuse.

[Materials]

Pad : absorbent cotton, polyethylene

Adhesive tape : polyester non-woven fabric, acrylic adhesive

[Purpose of Use]

Wound protection

[Product Description]

Sterile wound

[Usage]

Clean the wound and apply the pad to it, fix with tape.

[Cautions for Use]

- 1) Clean the wound and wound surrounding, make sure the applied area is dry and the pad is in clean condition.
- 2) For users with sensitive skin, do not apply on the same position repeatedly. Change the applied position 1-2 times a day.
- 3) Replace the dressing with a new one when it is wet or soiled.
- 4) Do not apply the adhesive to wound.
- 5) Stop using when any negative effects like rash, swelling, redness occur.
- 6) Users who has adhesive allergy should avoid using this product.
- 7) Use immediately after opening the package.

[Cautions for Storage and Handling]

1. Storage

Avoid direct sunlight and keep away from water. Keep in a clean and dry place.

2. Shelf Life

Unopened up to 5 years.

[Packaging]

- No. 50 (50mm × 80mm) 50 pcs per box
- No. 60 (60mm × 100mm) 50 pcs per box
- No. 80 (80mm × 130mm) 30 pcs per box
- No.100 (100mm × 160mm) 30 pcs per box
- No.110 (110mm × 200mm) 20 pcs per box
- No.120 (120mm × 260mm) 20 pcs per box

[Distributor]

Name : Hakuzo Medical Corporation

Address : 2-4-9, Tokuicho, Chuo-ku, Osaka 540-0025 Japan

TEL : +816-6809-4319

Product No.	Type	Size	Qty/box	Qty/carton	JAN Code
3155001	No.50	5cm × 8cm(Pad 2cm × 4cm)	50 pcs	20 boxes	 4 958286 701219
3155002	No.60	6cm × 10cm(Pad 3cm × 6cm)	50 pcs	20 boxes	 4 958286 701226
3155003	No.80	8cm × 13cm(Pad 4cm × 9cm)	30 pcs	20 boxes	 4 958286 701233
3155004	No.100	10cm × 16cm(Pad 5cm × 10cm)	30 pcs	20 boxes	 4 958286 701240
3155005	No.110	11cm × 20cm(Pad 5cm × 15cm)	20 pcs	20 boxes	 4 958286 701257
3155006	No.120	12cm × 26cm(Pad 6cm × 20cm)	20 pcs	20 boxes	 4 958286 701264



Design Your Voice

Hakuzo

Add: 2-4-9, Tokuicho, Chuo-ku,
Osaka 540-0025, Japan

TEL: 06-6942-0451

FAX: 06-6945-0539

www.hakuzo.co.jp/eng/index.php