

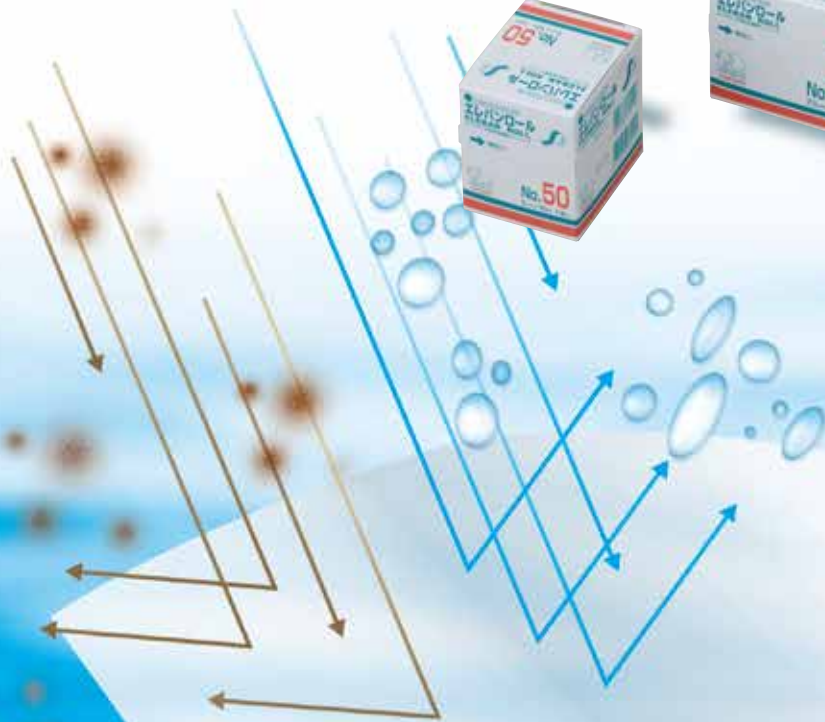
Transparent Dressing Tape



# ELEBAN ROLL

Roll type waterproof dressing made of transparent film.

Non Sterile  
Latex Free



Hakuzo Medical Corporation

# ELEBAN ROLL

Non Sterile  
Latex Free

## Product Features

### ● Stretchable Tape

Made of flexible, ultrashin transparent film which is highly stretchable and can be attached to joints.

### ● Waterproofness & Barrier Properties

External contamination risk can be decreased.

### ● Permeable Film

Water vapor permeable film prevents from feeling stuffy.

### ● Cost Effectiveness

Can be cut to desired size and used without wasting.

## Purpose of Use

- Wound dressing fixation
- Wound protection while showering
- Tube and catheter fixation

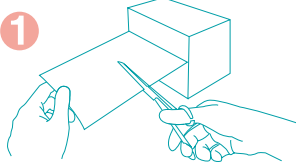
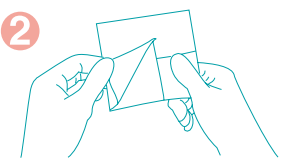
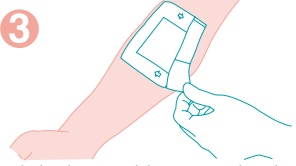
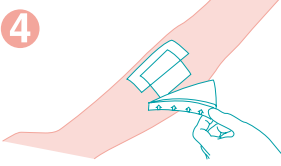
### ● Cautions on Use





- Clean and dry the skin before applying this product.
- Avoid contacting wounded sites directly.
- Make sure it is attached to skin firmly and securely when showering.
- Stop using when any negative effects like rash, pruritus or redness occur on skin.
- Do not apply this product for long period of time.
- Do not stretch the tape when applying.

### ● Cautions on Storage

- Keep dry and avoid direct sunlight. Keep in a cool and clean place.

## User Manual

<p><b>1</b></p>  <p>Tear off the perforated strip to open, pull out the roll, and cut it to the desired length with scissors.</p>	<p><b>2</b></p>  <p>Remove one side of the release paper.</p>
<p><b>3</b></p>  <p>Apply the dressing while removing the other release paper. Press down the edge of the dressing carefully and attach it firmly.</p>	<p><b>4</b></p>  <p>Remove the protective films from the surface in the direction of the arrows.</p>

Product No.	Type	Size	Qty/box	Qty/carton	JAN Code
3155041	No.50	5cm×10m	1 Roll	20 boxes	 4 958286 701516
3155043	No.75	7.5cm×10m	1 Roll	20 boxes	 4 958286 701523
3155045	No.100	10cm×10m	1 Roll	20 boxes	 4 958286 701530
3155047	No.150	15cm×10m	1 Roll	10boxes	 4 958286 701547



Design Your Voice

**Hakuzo**

Add: 2-4-9, Tokuicho, Chuo-ku,  
Osaka 540-0025, Japan

TEL: 06-6942-0451

FAX: 06-6945-0539

[www.hakuzo.co.jp/eng/index.php](http://www.hakuzo.co.jp/eng/index.php)