

# Inspection Use Belts

## Heart & Tummy Belts



- 1 The gap between holes are small, enabling more precise adjustment.
- 2 With wider width, transducers can be positioned firmly.
- 3 Clean personal belts.

**2** pcs pack  
Pink 1pc  
Blue 1pc

Size: 60mm × 1.3m



### We heard your voices!

In summer, belts get wet easily.

It is difficult to wash and manage

It is unpleasant to share belts that have direct skin contact.

It is difficult to get rid of blood and gel stains

I feel bad for the pre-mothers for getting them using the recycled products.

### Heart & Tummy Belts can help!

Pre-mothers discomfort

Medical staffs' workload will be reduced

Infection risks will be reduced!

Heart & Tummy Belts is used to fix ultrasonic transducer and labor pain transducer on a pre-mother' s tummy



### Product Features

- ♥ The gap between holes are small, enabling more precise adjustment
- ♥ With wider width, transducers can be positioned firmly
- ♥ Belts come in 2 colors to prevent mixed-ups when they are tied in fro
- ♥ Belts are thin and materials used are high in breathability
- ♥ Clean personal belts
- ♥ Materials used are latex free

### How to use

Put the belt around the waist and make sure that the belt is not twisted. Position the belt by having the knob goes through the holes on the belt.

### Comments from meetuu


Heart & Tummy Belts was named by combining "baby' s heart beat" and "pre-mother' s labor pain beat" , Hakuzo wanted to emphasize the special bond between a mother and her baby. During every routine check-up, Heart & Tummy Belts will be used to witness the progress made by the baby, right until the baby is delivered.

### Handling attentions

- Please do not use detergent with fluoresent brightening agent.
- Avoid direct sunlight. Store in a cool and dry place.
- Keep the water temperature below 40° C. Please wash in delicate mode, or wash by hands. Please use a laundry net when washed in a washing machine.
- Please do not use chlorine bleach.
- Please do not iron.
- Please do not dry under the sun.



### Product Information

Product Code	Product Name	Specs	Packaging	Materials	JAN Code
3147588	meetuu Heart & Tummy Belts	60mm×1.3m	2pcs (Pink 1 pc) (Blue 1 pc)	Polyester 74% Polyurethane 26%	 4 958286 707211



### Hakuzo Medical Corporation

4-9, 2-chome, Tokuicho, Chuō-ku, Osaka  
540-0025, Japan  
Tel: 81-6-6809-4319 Fax: 81-6-6945-0539  
E-mail: hmc-trade@hakuzo.co.jp  
<http://www.hakuzo.co.jp/eng/>