


# MAMARAKU Compression Stockings

Spiral compression stockings designed for pregnant women.

Stockings with suitable level of compression can be worn for a long period of time.

**【Purpose】**  
To improve blood circulation and reduce lower leg swelling.

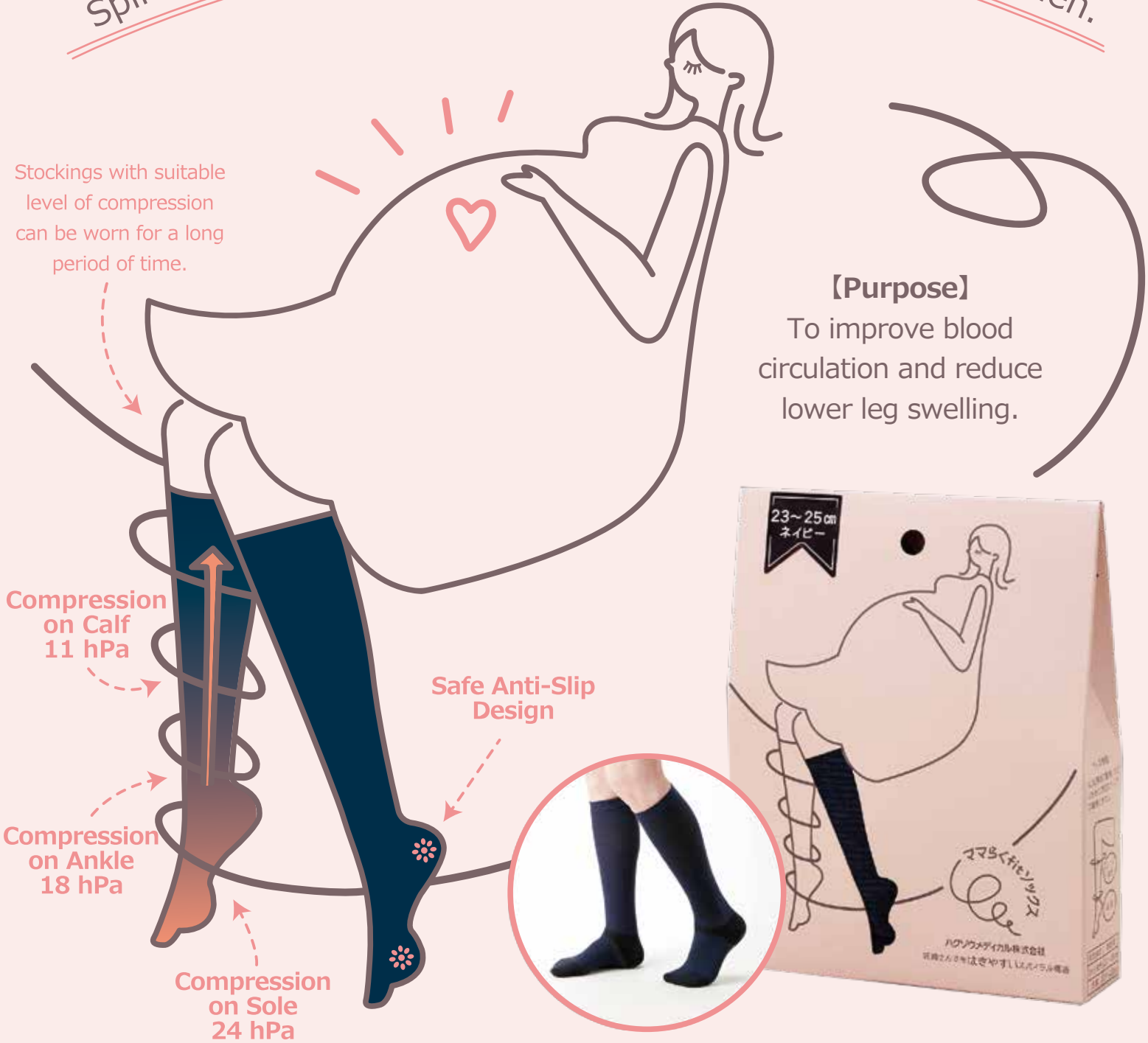

※Navy Blue

**Product Feature 1** Compression stockings with spiral support strip that provides suitable pressure without obstructing the flow of blood or fluid. According to our survey conducted in 2019, 90% of 32-36 week pregnant women think it is easy to wear and take off the product.  
 ※ The patent no. of the unique spiral strip is 5721972, owned by Okamoto Corporation.


**Product Feature 2** With moderate compression, the stockings can be worn for a long period of time.  
 Ergonomic compression ratio: **Sole : Ankle : Calf = 2.2 : 1.6 : 1**  
 ※ The ratio is recommended by The Japan Research Associate for Textile End-Users.

**Product Feature 3** Comes with anti-skid pods on the sole, safe to wear during pregnancy.



**Instructions** ※ Comes with markers on the inside of stockings to help distinguish left and right ones. (「左」 means left, 「右」 means right )

- ① Grab the top of stocking, roll it down toward the heel and put foot inside.
- ② Pull the stocking up to ankle and make sure it fits heel properly.
- ③ Pull the stocking up and make sure the left&right markers are in the middle position.
- ④ Pull the stocking up over the leg and smooth out any remaining wrinkles.

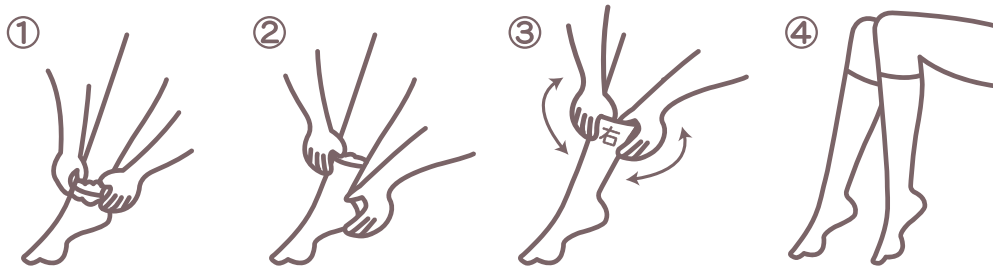

- Care Tips**
- ① Turn the stockings inside out and wash with a laundry bag.
  - ② Do not use bleach on this product.
  - ③ Rinse off the detergent with water thoroughly.
  - ④ Do not wash this product with white clothes.
  - ⑤ Scrubbing this product when it is wet may cause discoloration and lint balls.
  - ⑥ Do not dry with a clothes dryer.
  - ⑦ Dry the stockings in shade with toes pointing downward.
  - ⑧ This product may be wrinkled after washing. The function will not be affected.

**Size** The suitable size of calf and ankle is as follows.

Parts	Perimeter
Calf	32 ~ 38 cm
Ankle	19 ~ 23 cm



The size is printed on the labels.



Product No.	Specs.	Qty/carton	JAN Code
3000079	Navy Blue 21-23cm 1 pair/box	10 boxes x 4 inner cartons	4 958286 715919
3000080	Navy Blue 23-25cm 1 pair/box	10 boxes x 4 inner cartons	4 958286 715926



*Design Your Voice*  
**Hakuzo**

Add: 2-4-9, Tokui-Cho, Chuo-ku,  
 Osaka 540-0025  
 TEL: +816-6809-4319  
 FAX: +816-6942-0496  
 www.hakuzo.co.jp