

## Postpartum Seat Cushion

# MAMARAKU U Shaped Cushion

Pressure relief U shaped cushion with superior breathability.



Most of mummies suffer from vaginal pain and rectal prolapse after vaginal delivery.

To improve the quality of mummies' life, we have developed this postpartum seat cushion with NCGM and NCN.

Made from  
pressure relieving  
and breathable  
materials.

V-Lap  
Nonwoven Fabric

Spongy Fiber

3 colors of cushion covers are available



Hakuzo Medical Corporation

**Product Features**

2 kinds of materials, **Spongy Fiber** and **V-Lap Nonwoven fabric**, which are pressure relieving and breathable are being used.

## Spongy Fiber

Material : Polypropylene 100%

Features : High elasticity and rebound, superior breathability, light weight and heat resistance. Can be cleaned and disinfected by 0.5w/v% (5,000ppm) sodium hypochlorite, ethanol, isopropanol, etc.




**Hydrophobicity**

Can be washed by water and dry fast. Can be disinfected by different types of disinfectant.



**Breathability**

Spongy structure composed of 95% of air.

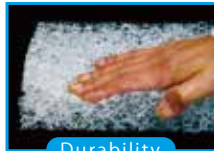

**Durability**

Can tolerate the compression tests for 80,000 times.

## V-Lap Nonwoven Fabric

Material : Polyester fiber

Features : High breathability, light weight, heat resistance. Can be washed in washing machine.



**V-Lap Nonwoven Fabric**

**Traditional Nonwoven Fabric**



V-Lap nonwoven fabric is the perfect material for cushion.

**Handling Methods**

- Washing may cause shrinkage.●Can be washed in washing machine up to 40°C●Wash with a laundry net.●Dry in shade after dewatering
- Cover the fabric with a cloth and iron at low temperature●Set the upper temperature limit to 80°C or less when using a dryer.●Use neutral detergent.●Can be disinfected by sodium hypochlorite: wash with warm water, rinse with 0.01 to 0.02 w/v% (100 to 200 ppm) sodium hypochlorite for 5 minutes.


Released in the 59th Annual Meeting of Japan Society of Maternal Health

**【Test Regarding the Pressure Relieving Function】**


<The test results are based on 55kg females > Pressure (kPa)

	minimal pressure	maximal pressure	average pressure	standard deviation
Mamaraku	0.00	16.57	4.26	±4.52
Urethane cushion	0.00	20.88	4.45	±5.38
Rubber cushion	0.00	25.38	4.59	±5.94


Mamaraku U-shaped cushion

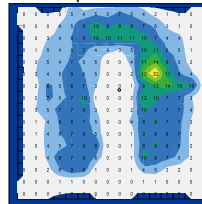

Urethane donut seat cushion

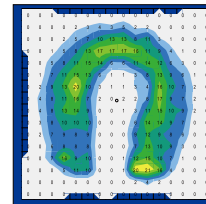

Rubber donut seat cushion

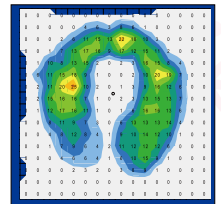

**Precautions**

- Keep zipper face down when using.
- Avoid direct sunlight and keep in a dry and cool place.
- Keep away from the reach of children.

## Specifications

Product No.	Product Name	Colors	Qty/box	Box size/weight	Qty/carton	Carton size/weight	JAN code
2590130	Mamaraku U-shaped Cushion	Navy blue 	1 pc	W453×D460×H76mm 1,040g	15boxes	W1,200×D500×H500mm 17,7kg	 4 958286 714813
2590131		Brown 	1 pc	W453×D460×H76mm 1,040g	15boxes	W1,200×D500×H500mm 17,7kg	 4 958286 714820
2590132		Blue 	1pc	W453×D460×H76mm 1,040g	15boxes	W1,200×D500×H500mm 17,7kg	 4 958286 714837



Design Your Voice

# Hakuzo

Add: 2-4-9, Tokuicho, Chuo-ku, Osaka 540-0025, Japan

TEL: 06-6942-0451

FAX: 06-6945-0539

www.hakuzo.co.jp/eng/index.php