

.....When to Wear?.....

Once Lochia Stops

To Help Mommies
Get Back in Shape

**Postpartum
Corset**

MOMMY SLIM

DOUBLE



**Firm
Compression**

Support and flatten tummy firmly

3-Step Fastening System!

Postpartum Recovery Garments	Delivery	Once Lochia Stops	2 Months after Delivery
Mother Belt (Around Pelvis)	Pink/White/Light		
Mommy Slim (Around Abdomen)	SINGLE Gentle Type	
	DOUBLE Firm Type	

Suitable Waist Size

M~L 62~84cm

L~LL 72~94cm

MADE IN JAPAN

※ This is the recommended time to wear for reference only. Adjust according to one's physical condition.

3-Step Fastening System Supports Tummy Firmly and Securely!



To Help Mommies
Get Back in Shape

Postpartum Corset

How to Put on




1 Unfasten **B** support belt, **C** twin belt, and position the **A** main belt at elbow height.

2 Adjust the tightness and fasten **A** main belt.

3 Fasten **B** support belt.



4 Fasten **C** twin belt.
Finished.

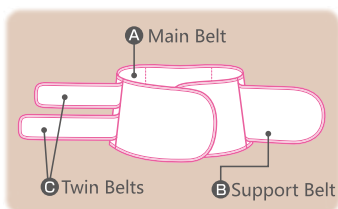

Wearing Tips

- The suitable tension differs from person to person. Choose the tension that one is most comfortable with.
- If direct contact with skin cause irritation, wear something in between.

Firm Compression

Comes with twin belts and can be adjusted easily to provide suitable compression.



3-Step Fastening

The firmness of the corset is largely increased by using 3-step fastening system.

High Quality

Made of medical-grade fabric and can support abdomen firmly and securely.

Specifications

Product No.	Product Name	Size	Suitable Waist Size	Qty/box	JAN Code
1490526	meetuu Mommy Slim Double	M~L	62~84cm	1 pc/box	 4 958286 711737
1490527		L~LL	72~94cm	1pc/box	 4 958286 711744



Design Your Voice

Hakuzo

Add: 2-4-9, Tokuicho, Chuo-ku,
Osaka 540-0025, Japan

TEL: 06-6942-0451

FAX: 06-6945-0539

www.hakuzo.co.jp/eng/index.php