

.....When to Wear?.....

Once Lochia Stops

To Help Mommies
Get Back in Shape

Postpartum
Corset

MOMMY SLIM

SINGLE



Gentle
Compression

Comfortable and easy to wear
1-step fastening corset♪

Postpartum Recovery Garments	Delivery	Once Lochia Stops	2 Months after Delivery
Mother Belt (Around Pelvis)	Pink/White/Light		
Mommy Slim (Around Abdomen)	SINGLE Gentle Type	DOUBLE Firm Type

Suitable Waist Size	
S~M	60~75cm
M~L	70~90cm
L~LL	80~105cm

MADE IN JAPAN

※ This is the recommended time to wear for reference only. Adjust according to one's physical condition.

Flatten Tummy with Gentle Compression!
Comfortable 1-Step Fastening Corset



To Help Mommies
Get Back in Shape

Postpartum Corset

How to Put on



1 Make sure the hook and loop fastener is outside and position the corset at elbow height. (The curved side should be positioned at the bottom.)

2 Adjust to the suitable tightness and fasten.

Wearing Tips

- The suitable tension differs from person to person. Choose the tension that one is most comfortable with
- If direct contact with skin cause irritation, wear something in between.

Gentle Compression

3D ergonomic cutting design makes it fit perfectly and gives moderate compression on the tummy.

1-Step Fastening

Comes with 1-step fastener and can be put on and adjusted to comfortable tightness easily. Suitable for busy mommies.

High Quality

Made of medical-grade fabric and can support abdomen firmly and securely.

Specifications

Product No.	Product Name	Size	Suitable Waist Size	Qty/box	JAN Code
1490601	meetuu Mommy Slim SINGLE	S~M	60~75cm	1 pc/box	4 958286 705613
1490602		M~L	70~90cm	1 pc/box	4 958286 705620
1490603		L~LL	80~105cm	1 pc/box	4 958286 705637



Design Your Voice

Hakuzo

Add: 2-4-9, Tokuicho, Chuo-ku,
Osaka 540-0025, Japan

TEL: 06-6942-0451

FAX: 06-6945-0539

www.hakuzo.co.jp/eng/index.php