

From the
date of delivery


Suitable for
outdoor use

Thin Type



To tighten loose pelvis Mother Belt

LIGHT

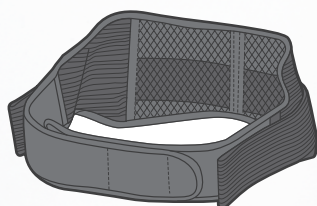

Thin Type
Natural
look



Mesh material is used
for better ventilation
Keeping skin dry



2 belts attached
Easy adjustment



6cm width belt
Pelvis fitting

COLOR
Black


Best care after delivery	Delivery	Once lochia stops	2 months after delivery
Mother Belt (around pelvis)	Pink/White	Daily standard type	
	LIGHT	Outdoor use type	
Mama Slim (around waist and tummy)	SINGLE/DOUBLE	

Suitable hip size

S~M	60~85cm
M~L	85~105cm
L~LL	105~120cm

MADE IN JAPAN

Light type pelvis tightening belt

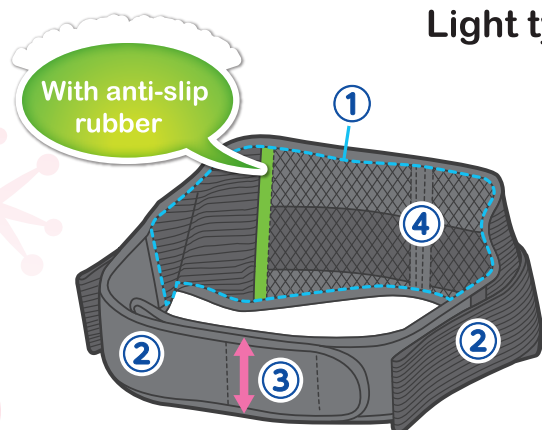

Thin Type
Suitable for outdoor use

① Light, thin type

This product will not look obvious when worn under your clothes, it is recommended when going outdoor. Mesh is the main material, giving the belt best breathability possible.

② Easy adjustment

The two belts give more accurate adjustments.

③ Comfortable belts


Belts with 6cm width fit well, and the presence of the belts can be easily felt. The pressure on the stomach is minimised and discomfort is hardly felt.

④ Firm design

With the back supporter, the firmness of the belt is largely increased.

Wearing tips

- The suitable tension differs from person to person. Choose the tension that one is comfortable with.
- If direct contact with the skin causes irritation, try to wear something in between.

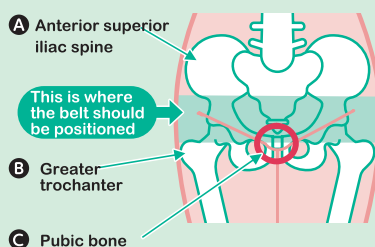

A Anterior superior iliac spine

This is where the belt should be positioned


B Greater trochanter

C Pubic bone

How to put on



Make sure that the supporting belt is outside and unfasten the supporting belt. Look for **A** anterior superior iliac spine and **B** greater trochanter. Put the belt around the buttock, and in between **A** & **B**.

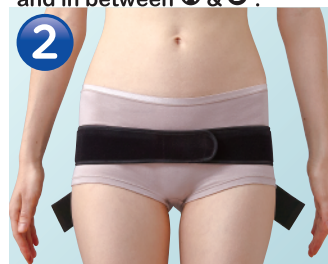

Make sure that the front belt overlaps with **C** pubic bone.






Hold the supporting belts and pull them forward. Fasten them.



Finished.

Product Information

Product code	Product name	Specs	Packaging (size/weight)	JAN code
1490520	meetuu Mother Belt Light	S~M Black	1pc/box W118×D35×H215mm About 130 g	 4 958286 709567
1490521		M~L Black	1pc/box W118×D35×H215mm About 140 g	 4 958286 709574
1490522		L~LL Black	1pc/box W118×D35×H215mm About 155 g	 4 958286 709581



Design Your Voice

Hakuzo

Add: 2-4-9, Tokuicho, Chuo-ku
Osaka 540-0025, Japan

TEL: 06-6942-0451

FAX: 06-6945-0539

www.hakuzo.co.jp/eng/index.php