


SUGU Mother Belt

Non-woven Disposable Type Pelvic Belt

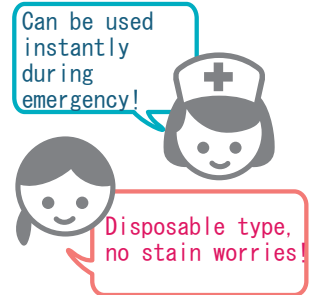

Product Features

- ♥ Non-elastic type with 6cm width, able to tighten the pelvic bones firmly.
- ♥ The belt can be fixed anywhere.
- ♥ Light and high in breathability.
- ♥ Disposable type that is hygienic.

Attentions:

- Please tighten in between anterior superior iliac spine and greater trochanter.
- Please change to Mother Belt for longer period of usage.


When to use

Please use **right after delivery**



Suitable hip size

M~L	85cm-105cm
L~LL	105cm-120cm

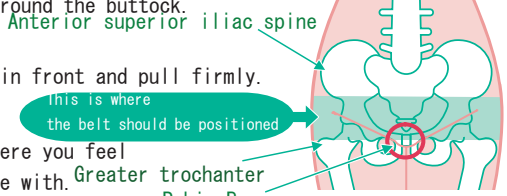How to use

- 1 Keep the fluffier surface outside, search for the anterior superior iliac spine and greater trochanter, and fix the belt in between them by going around the buttock.

- 2 Cross the belt in front and pull firmly.
- 3 Fix the belt where you feel most comfortable with.
 Greater trochanter
 Pubic Bone

Product Information

Product Code	Product Name	Specification	Packaging	Materials	JAN Code
3108201	meetuu SUGU Mother Belt	M ~ L (Width60mm × length1.35m)	1 pc/pack	Main Materials : Polyster 70% Rayon 30%	 4 958286 709017
3108202		L ~ LL (Width60mm × length1.55m)			 4 958286 709024



Hakuzo Medical Corporation

4-9, 2-chome, Tokuicho, Chuo-ku, Osaka
 540-0025, Japan
 Tel: 81-6-6809-4319 Fax: 81-6-6945-0539
 E-mail: hmc-trade@hakuzo.co.jp
<http://www.hakuzo.co.jp/eng/>