

HAKUZO

Refreshing Oral Sponge

Paper Shaft

Firm paper shaft is suitable for removing hardened plaque. Dissolvable paper shaft is environmentally friendly.



Portable
Small
Pack

Plastic Shaft

Waterproof plastic shaft can be soaked in water or solution. Suitable for the care receivers with high saliva secretion or who tends to bite the swab during the oral care.



Comes with 6 notches

Comes with 6 notches in the sponge head that improve the sponge's cleaning performance.

- ① The sponge can remove contaminants adhered to uneven surfaces.
- ② The oral cavity can be cleaned gently with the fine textured sponge tip.

【Instructions】

- To apply moisturizing gel.



- Place an adequate amount of the gel on a plate or a glove. Spread the gel evenly on the sponge.
- Use the sponge to apply the gel on the surfaces of oral cavity.

- To clean the oral cavity.



- Move the sponge back and forth while rotating the sponge to clean the gums and the tongue accordingly.

Scan the QR code to watch the instructional video.



Stand up pouch with 50 swabs is convenient to carry around in the facility.
Small pouch with 10 swabs is suitable for home nursing care.

Paper shaft / 50 swabs



Plastic Shaft / 50 swabs



Plastic shaft / 10 swabs



Specifications

Product Code	Packaging	Qty/box	Qty/carton	JAN Code
1240973	Paper shaft, 50 swabs/pouch	6 pouches/box	6 boxes	4 958286 717920
1240976	Plastic shaft, 50 swabs/pouch	6 pouches/box	6 boxes	4 958286 717937
3118060	Plastic shaft, 10 swabs/pouch	20 pouches/box	6 boxes	4 958286 718637



Design Your Voice
Hakuzo

Add: 2-4-9, Tokui-cho, Chuo-ku,
Osaka 540-0025, Japan

TEL: 06-6942-0451

FAX: 06-6945-0539

www.hakuzo.co.jp/eng/index.php