

SINGLE USE

MEDICAL DEVICE



Design Your Voice

Hakuzo

ELEBAN PRESTAT[®] MINI TH

Injection Plaster

CE 0123

To protect the puncture site after intravenous injection, vaccination, blood drawing with tape, and absorbent pad




STERILE EO 100 PCS

【Size】

- Tape size : 36x36mm
- Pad size : Φ 15mm

【Materials】

- Tape: Non-woven fabric, acrylic adhesive
- Release Paper: Paper, silicone resin
- Pad: Non-woven fabric

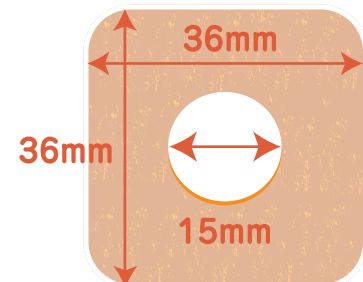

Product Feature

① Acrylic Adhesive

Hypoallergenic acrylic adhesive is being used.
The risk of skin irritation can be minimized.

Adhesive Side




Low risk of skin irritation

Product Feature

② Folded Edge

Comes with folded edge that gives easier handling.

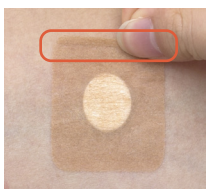

Folded Edge

Easy Handling

Product Feature


③ Release Paper Attached to Pouch

With the release paper attached to the pouch, the operation can be easier and the rubbish can be reduced.

The release paper is attached to the pouch.

- 3 steps to apply to wound.
- Rubbish reduction.
- Hygienic handling.

Before peeling off

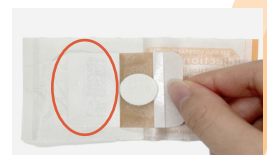

After peeling off



Convenient and hygienic.

The release paper is attached to the pouch.

ELEBAN PRESTAT[®] MINI TH Injection Plaster


How to use




Open the packaging, hold the plastic tab and tape on the edge, pull it out from the packaging.



Press the pad on the wound, with the plastic tab on.

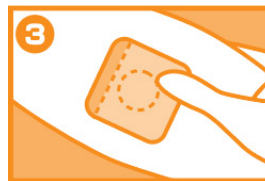

Pull out the plastic tab, press the tape evenly

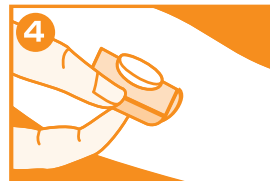

To take off, pull the tape from the edge.

ELEBAN PRESTAT[®] MINI TH Injection Plaster

Sterile and Single Use

【 Warning and Precaution 】

- Do not use if the package is damaged.
- Sterile and Single Use (Do not reuse plaster)
- Stop using when skin irritation occurs from acrylic adhesive.
- Throw away in infectious waste after usage.

【 Materials 】

Tape : Non-woven Rayon Adhesive : Acrylic adhesive
Pad : Non-woven fabric
Release paper: Paper, Silicone resin

【 Intended Use 】

To protect the puncture site after intravenous Injection, vaccination and blood drawing with tape and absorbent pad.

【 How to use 】

Clean the wound, open the pouch, hold the tape with release paper and apply the pad on the wound. Then fix with the tape. Do not reuse plaster .

【 Precaution 】

1. Dry it after cleaning the wound and it's surrounding.
Do not contaminate the pad.

2. For users with sensitive skin, do not apply on the same site repeatedly. Change the position 1-2 times per day.
3. Stop using and take proper action when skin lesion or wound infection occurs.
4. Do not apply adhesive on the wound.
5. Stop using when skin irritation occurs.
6. Avoid using this product if allergic to acrylic adhesive.
7. Use immediately after opening the package.

【 Storage Condition and Expiry date 】

1. Storage

Avoid direct sunlight and wet environment.
Keep in a dry, clean and cool place.

2. Expiry date

Refer to the package.

【 Manufacturer and Owner 】

Hakuzo Medical Asia Co., Ltd.
390/18 WHA Chonburi Industrial Estate 2 Moo 2,
Khao Khansong, Sri Racha, Chonburi 20110
<https://www.hakuzoasia.co.th/> TEL : 033-136103

Product code	Tape size	Pad size	Qty (pcs/box)
8300023	Size M 36 x 36mm	Φ15mm	100

Instructions video ⇒

※The video shows the usage procedure using 'Eleban Prestat[®] Mini TH

